



## Nylon Self-Seal Pouch INSTRUCTIONS FOR USE:

1. Select appropriate size pouch and insert instruments or cassette inside.

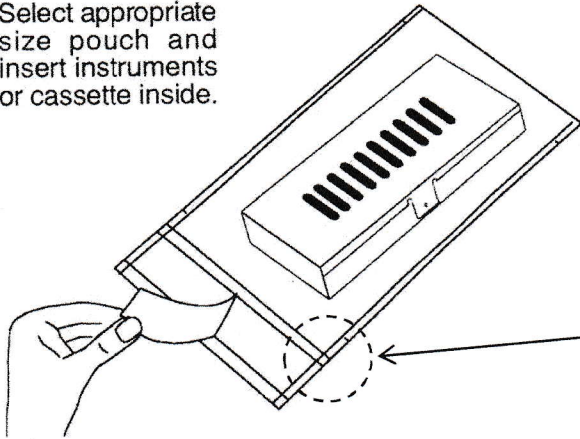

2. Peel away the adhesive tab, fold and secure by pressing firmly from side to side.

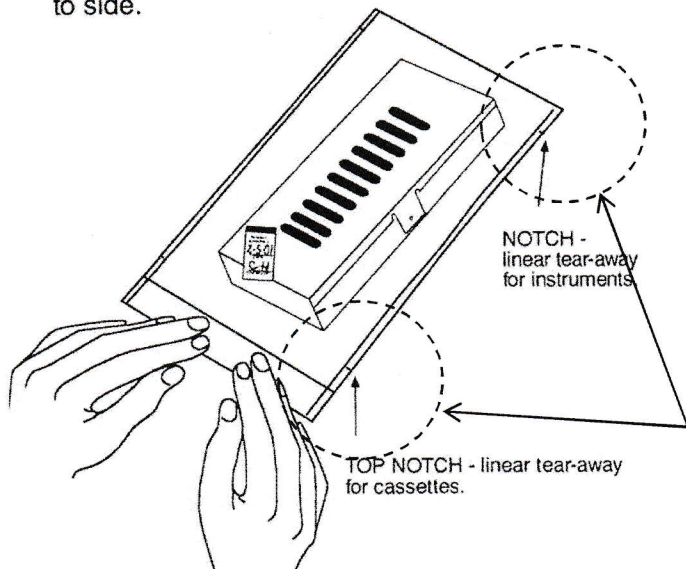

### Special Note:

Before processing, a chemical indicator label should be added to verify processing.

After processing, open pouch via the NOTCH located on either side. Cassettes should only be opened using the TOP NOTCH.

## Nylonové samolepící sáčky

### Návod k použití

1. Vyberte správnou velikost sáčku a vložte nástroje nebo kazetu dovnitř.
2. Odlepte papírek na lepicí vrstvě, přeložte a zalepte stiskem ze středu směrem na stranu.  
POZOR: ohněte podél lepicí vrstvy tak, abyste překryli mezeru z obou stran lepidlem!
3. Nalepte procesový indikátor.  
Indikátor při sterilizaci změní barvu z šedé na černou.
4. Sáček otevřete zatažením za zářez na okraji.



SPSmedical Supply Corp. • 6789 W. Henrietta Road • Rush, NY 14543  
(585) 359-0130 • (800) 722-1529 • Fax: (585) 359-0167  
E-mail: [info@spsmedical.com](mailto:info@spsmedical.com)  
Visit our website: [www.spsmedical.com](http://www.spsmedical.com)